# LELO

## **LUNA BEADS™**

USER MANUAL & WARRANTY INFORMATION

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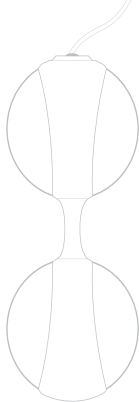
Congratulations on acquiring LUNA Beads™. Before getting more closely acquainted with them, please take a moment to look through this user manual.

## REGISTER WARRANTY

To register your warranty, find your warranty number on the LELO Authenticity card provided. Then visit **LELO.com** and click on **REGISTER** in the top right side of the page. This allows you to confirm the authenticity of your purchase and protect your investment.

## CONTENTS

|   |    |
|---|----|
| OVERVIEW.....                                   | 4  |
| WARRANTY TERMS & CONDITIONS.....                | 5  |
| COMPOSITION.....                                | 6  |
| INSERTING LUNA BEADS™.....                      | 8  |
| POSITIONING.....                                | 9  |
| USING LUNA BEADS™ AS PART OF KEGEL ROUTINE..... | 10 |
| CLEANING, STORAGE AND SAFETY.....               | 11 |
| SPECIFICATIONS.....                             | 12 |

## OVERVIEW

Every woman is different. That's why LUNA Beads are available in two size variations: **Classic** and **Mini**. LUNA Beads Mini have been specially designed to offer a perfect fit for women under 30 who have never experienced childbirth, while LUNA Beads are recommended for over-30's and those who have experienced childbirth.

Each set of LUNA Beads consists of a silicone harness and four weighted beads: two petal pink 28-gram beads (one foundation bead with retraction leash and one crown bead) and two powder blue 37-gram beads (one foundation bead with retraction leash and one crown bead). When worn individually or combined using the silicone harness, LUNA Beads offer an easy, effective and effortless way to optimize your pelvic floor workout.

LUNA Beads Noir are the sensual take on this bestselling product, designed to be used during foreplay or as your own little secret on a night out with your partner. They consist of 2 weighted beads (37 grams) in Mini Size that can be combined with a black silicone harness. The details in this manual primarily concern the use of LUNA Beads as a Kegel Exercising Aid, yet if bought for pleasure there are more detailed tips and tricks available on [LELO.com](http://LELO.com) on how to get the most from your purchase.

Please note these are guidelines only. For example, LUNA Beads Mini are often preferred by many women starting to go through menopause, while the original LUNA Beads may be preferred by younger women just starting their Kegel routine.

**Always consult your doctor before using LUNA Beads during pregnancy.**

## WARRANTY TERMS & CONDITIONS

### 1-YEAR WARRANTY

LELO warrants this product for a period of ONE (1) YEAR, after the date of original purchase, against defects due to faulty workmanship or materials. If you discover a defect and notify LELO during the warranty period, LELO will, at its discretion, replace the product free of charge.

The warranty covers working parts that affect the function of the product. It does NOT cover cosmetic deterioration caused by fair wear and tear or damage caused by accident, misuse or neglect. Any attempt to open or take apart the product (or its accessories) will void the warranty.

Claims under warranty must be supported by reasonable evidence that the date of the claim is within the warranty period. To validate your warranty, please keep your original purchase receipt together with these warranty conditions for the duration of the warranty period.

To claim your warranty, fill in your details under CUSTOMER CARE on [LELO.com](http://LELO.com). You will be sent a Return Merchandise Authorization (RMA) number which should be returned with the LELO product to your nearest LELO office carriage paid. Shipping costs are non-refundable. This undertaking is in addition to consumer's statutory rights and does not affect those rights in any way.

### 10-YEAR QUALITY GUARANTEE

As a continuation of the terms of LELO's full one (1) year warranty, LELO's ten (10) year quality guarantee makes owners eligible to purchase a new product from [LELO.com](http://LELO.com) for 50% off the listed price.

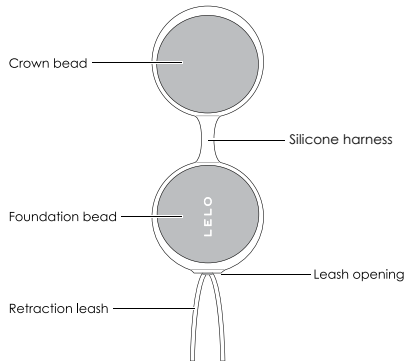
See the CUSTOMER CARE section of [LELO.com](http://LELO.com) for Terms & Conditions.

## COMPOSITION

The foundation beads can be used individually or combined with a crown bead using the silicone harness.

- 1. Thread the retraction leash, on the foundation bead of your choice, through the leash opening in the foundation bead nook on the silicone harness.**
- 2. Push the foundation bead into place, making sure the edges of the nook slot firmly into place and trace the contours of the harness guide imprint on the foundation bead.**
- 3. Push the crown bead into place, making sure the edges of the crown nook slot firmly into place and trace the contours of the harness guide imprint on the bead.**

Depending on your level of vaginal and pelvic floor fitness, you may wish to start out with a single foundation bead and gradually progress to compositions involving two beads and more weight. The most important thing is that the composition you opt for feels comfortable.

## INSERTING LUNA BEADS™

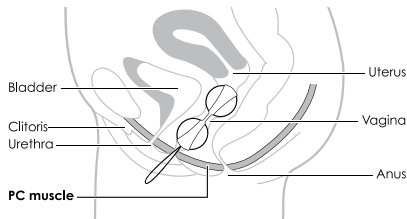
Inserting LUNA Beads is not dissimilar to inserting a tampon. As LUNA Beads have a wider girth, however, some additional tricks apply.

1. **To facilitate easy insertion, apply a small amount of water-based lubricant on the leading edge of the bead(s) you intend to use. Avoid using too much, as this will make the bead(s) slippery and difficult to hold in place.**
2. **Lie on your back and introduce the leading end of the bead(s) into your vagina with a firm but gentle push. If you experience difficulties, raise your pelvis so that gravity can assist you. Some women find that a standing position, with one leg raised, works better for them.**
3. **If you are using two beads and the silicone harness, insert the crown bead first. Due to the gap between the two beads, once the crown bead is inserted the vaginal muscles may try to push it out before you can insert the foundation bead. Take a deep breath and relax.**

## POSITIONING

The bead(s) should be positioned so that the lower edge of the foundation bead is at least 2 cm inside you. The most important thing is that the foundation bead rests above your pelvic floor muscle. The exact distance from the vaginal opening to the pelvic floor muscle differs from woman to woman, but the bead(s) should be positioned comparably to where a tampon would normally be positioned.

**Always allow the retraction leash to protrude slightly from the vagina so as to facilitate easy removal of the bead(s).**

## USING LUNA BEADS™ AS PART OF KEGEL ROUTINE

If you intend to use LUNA Beads primarily for exercise purposes, it is important that you incorporate it into your daily routine and adhere to a fixed program.

Depending on your level of vaginal and pelvic floor fitness, it may take up to 12 weeks of daily 30-minute Kegel exercising using LUNA Beads to achieve noticeable results.

At the outset of using LUNA Beads as part of your Kegel routine, you may feel slightly uncomfortable, as your body will be very aware of the beads. You may also find that beads tend to slip out after only a few minutes. If this happens, do not worry, and persist with the program. As your muscles become stronger, they will automatically hold the beads in place.

When you walk, run, swim or otherwise move about, the motion causes the inner balls to move, creating a subtle kinetic vibe stimulating the vaginal muscles.

Even when you are standing still, gravity will instigate spontaneous pelvic floor muscle contractions as your muscles work to prevent the beads from slipping out.

## CLEANING, STORAGE AND SAFETY

The lifespan of your LUNA Beads will be greatly increased with proper maintenance and care.

**LELO recommends that you NOT use a silicone-based lubricant with your LUNA Beads, as this could make the silicone harness permanently tacky. A water-based lubricant is the safest option. If you wish to use a silicone-based lubricant, perform a patch-test first. Never use massage oil or hand cream as lubricants.**

Always disassemble and clean each part of your LUNA Beads thoroughly before and after each use. Clean using warm water and antibacterial soap, rinse with clean hot water and pat dry with a lint-free cloth or towel. Never use cleansers containing alcohol, petrol or acetone.

Avoid leaving your LUNA Beads in direct sunlight and NEVER expose it to extreme heat.

Store your LUNA Beads in a dust-free place and keep it away from toys of other materials.

**IMPORTANT: For Adult Use Only.** Those who may be pregnant, have a pacemaker, diabetes, phlebitis or thrombosis should consult with a medical professional regarding the risk of blood clots before using any personal massage product.

## SPECIFICATIONS

|              |  |
|--------------|--|
| MATERIALS    | Body safe PC ABS / silicone  |
| FINISH       | Glossy   |
| HARNESS SIZE | Classic: 89 x 35 x 16 mm<br>Mini: 73 x 31 x 15 mm<br>Noir: 73 x 31 x 15 mm               |
| BEAD SIZE    | Classic: Diameter 35 mm<br>Mini: Diameter 29 mm<br>Noir: Diameter 29 mm                  |
| WEIGHTS      | 2 x 28 grams (petal pink)<br>2 x 37 grams (powder blue) <hr/> Noir: 2 x 37 grams (black) |

**Disclaimer:** Users of this product do so at their own risk. Neither LELO nor its retailers assume any responsibility or liability for the use of this pleasure object. Further, LELO reserves the right to revise this publication and to make changes from time to time in the contents hereof without obligation to notify any person of such revision or changes.

**Model may be changed for improvements without notice.**

© 2013 LELOi AB. ALL RIGHTS RESERVED.  
DESIGNED AND DEVELOPED BY LELO SWEDEN.  
MANUFACTURED BY LELO IN CHINA.

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LELO IS AN INTERNATIONALLY REGISTERED TRADEMARK OF LELOi AB.
REGISTERED THROUGH SUBSIDIARIES IN THE U.S.A AND OTHER COUNTRIES.